

BP37™

Fitting Instructions BABYPOD - METAL CLIP




IT IS IMPORTANT THAT THE FITTING OF THE METAL CLIP TO AHT BABYPODS USING THE FITTING KIT IS PERFORMED BY A COMPETENT ENGINEER / TECHNICIAN.



Parts Supplied:

Fitting Kit (Part No. BP37-010) contains:

- Metal Clip
- 2 x M4 screws,
- 4 x M4 washers
- 2 x M4 locking nuts
- Template

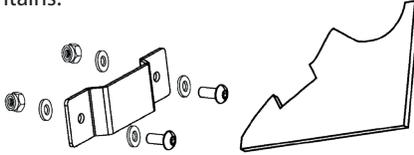

Tools Required (not supplied):

- Powered Drill
- 4mm Drill Bit suitable for metal
- 2.5mm Allen Key
- Size 7 Hex spanner
- Masking tape

How to attach the Metal Clip

1. Remove any components from inside the BabyPod shell.
2. The recommended position for mounting the Controller (ensuring this does not interfere with the correct functioning of the BabyPod stretcher straps and buckles), the clip is positioned 11cm from centre of clip to inside edge of stretcher buckle recess that is located at rear of the BabyPod, parallel to the contours of the BabyPod. (See figure 1)

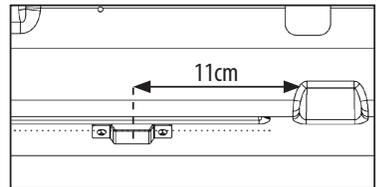


figure 1

Note: The profile of the Controller has been designed to match the contours of the BabyPod shells.

3. To protect BabyPod shells and prevent slippage whilst drilling, place masking tape over intended mounting area.
4. Place Metal Clip inside template and hold up to the side of the BabyPod shell, ensuring clip is parallel to the contours of the BabyPod. (See figure 2)
5. Mark hole positions.
6. Drill holes with 4mm drill bit. Wear appropriate PPE whilst drilling. (An assistant may be required to hold the BabyPod steady, whilst drilling.)
7. Remove masking tape and clean up any debris.
8. Attach Metal Clip to side of BabyPod using the supplied screws and locking nuts. (figure 3)
9. Attached metal clip on BabyPod / NeoPod. (figure 4)
10. Check Controller can be connected securely to metal clip. Refer to: BP37 - Fitting Instructions for BabyPod Variant.

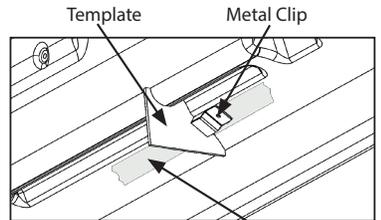


figure 2

Masking Tape

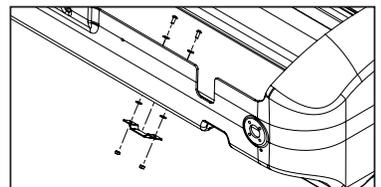


figure 3

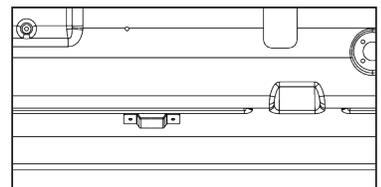


figure 4



Manufactured in France on behalf of:



Advanced Healthcare Technology Ltd.
Theobalds Enterprise Centre, Platinum Way, Waltham Cross, Herts., EN8 8YD, United Kingdom
Tel: +44(0)1992 535933 Email: sales@babypod.com
www.babypod.com



Advanced Healthcare Technologies s.r.o.
Varšavská 715/36, 120 00 Prague 2, Czech Republic



BP37™ is a patented product.
BP37™ is a trademark of Advanced Healthcare Technology Ltd.