

BP37™

Matrix - Instructions for Use




IMPORTANT: THESE INSTRUCTIONS, MUST BE READ IN CONJUNCTION WITH THE BP37 INFANT TEMPERATURE MAINTENANCE KIT IFU.




Part No. BP37-008 - Matrix (Pack of 10)

Store in a dry area, away from direct sunlight and heat sources, at temperature 5°C - 35°C.


Check Matrix is not damaged. If damaged, **do not use**.

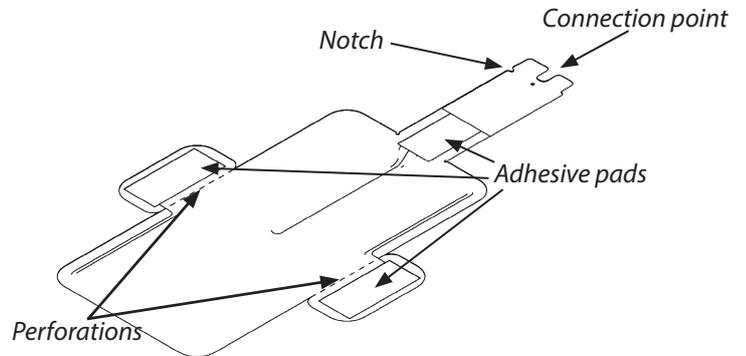


Remove Matrix from pouch.

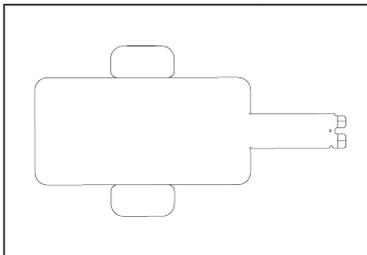


Place Matrix on vacuum mattress / mattress, non-woven side up.

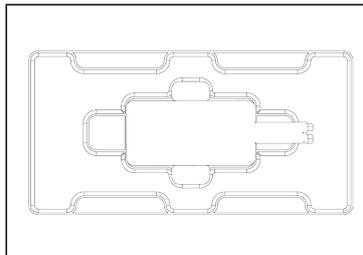


Peel off release paper from adhesive pads.

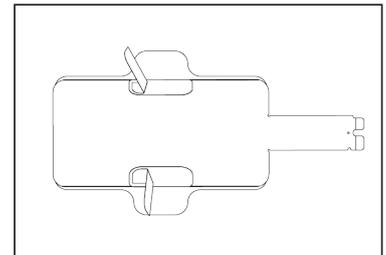

Rub down wings and tail.

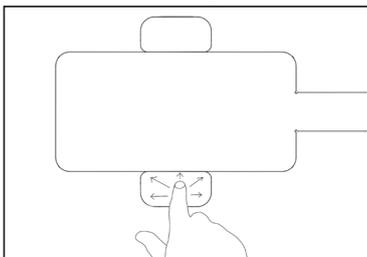

Matrix ready for use.

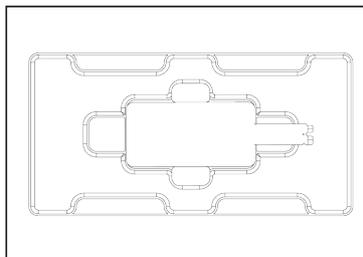

Note: Place baby on non-woven side up.

Note: Matrix are single patient multiple use.



Manufactured in France on behalf of:



Advanced Healthcare Technology Ltd.
Theobalds Enterprise Centre,
Platinum Way, Waltham Cross,
Herts., EN8 8YD, United Kingdom
Tel: +44(0)1992 535933
Email: sales@babypod.com
www.babypod.com



Advanced Healthcare Technologies s.r.o.
Varšavská 715/36, 120 00 Prague 2, Czech Republic



BP37™ is a patented product.
BP37™ is a trademark of Advanced Healthcare Technology Ltd.