


BP37™

QUICK SET-UP GUIDE

PLEASE REVIEW THE IFU BEFORE USE

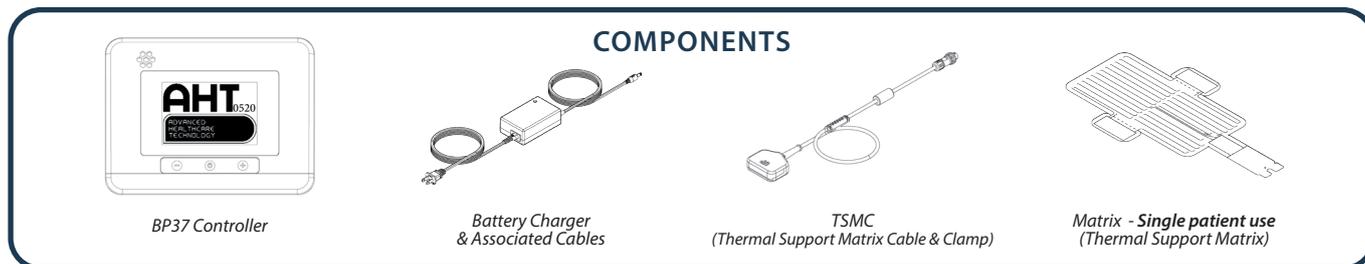

1. **Before Use:** Ensure the Controller has adequate charge, charge if required (figure 1).

2. Unscrew protective dust cover and connect TSMC to Controller (figure 2).

3. Remove Matrix from pouch and position on mattress.
Note: Do not put the baby on the plastic / silver side of the Matrix.

4. Secure Matrix to mattress by removing release paper from wings and gently rubbing wings down.

5. Connect TSMC to Matrix tail (figure 3).

6. Switch on Controller (Push and hold **⏻**).

7. Lay baby / infant on top of Matrix.

8. When thermal support is required, switch Controller to **37** Mode (Push and hold **+**).

9. If thermal support is not required, switch Controller to Standby (**SB**) (Push and hold **-**).

When thermal support is no longer required, remove baby from Matrix.

10. Switch Controller to Stand By Mode (**SB**), then turn off Controller.

11. Unclamp Matrix.

12. Tear off Matrix from mattress and remove remaining wings and dispose of.

13. Put Controller on charge as required (figure 1).

Note: If **37** Mode is activated when no infant present, it is recommended to place an insulating layer over the Matrix. This will help to achieve a surface temperature of 37°C (99°F) on the Matrix. Otherwise, dependent on ambient temperature, the surface temperature may not reach 37°C (99°F) until an infant is laid on the Matrix.

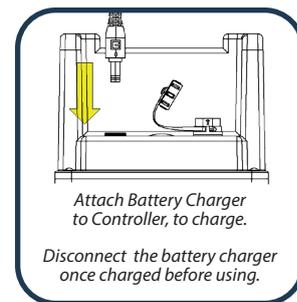


figure 1

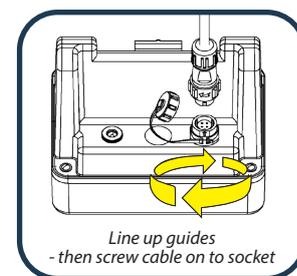


figure 2


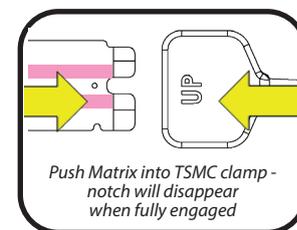


figure 3

1. On/Off Switch - Hold till beep
2. Switch to **37** Mode (thermal support) - Hold till beep
3. Switch to Stand By Mode **SB** (no thermal support) - Hold till beep
4. Battery Charge Indicator
5. Stand By Mode (**SB** - Controller in Stand By Mode)
6. **37** Mode (**37** - Controller in **37** Mode)
7. Ambient Temperature
8. Matrix Temperature
9. Activated Duration Time
10. Thermal Support activity
(Active ) (Inactive ) (Software safety ) (Hardware Safety )

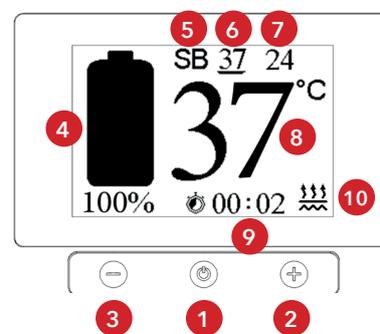

BP37™ is a trademark of AHT Ltd. BP37™ is a patented product.

Advanced Healthcare Technology Ltd.

Theobalds Enterprise Centre, Platinum Way, Waltham Cross, Herts. EN8 8YD, UK | www.babypod.com

